



Made in the United Kingdom



Sutherland Health Ltd
Unit 1 Rivermead Pipers Way
Thatcham Berkshire RG19 4EP
Tel: +44(0)1635 874488
Fax: +44(0)1635 877622
www.sutherlandhealth.com
info@sutherlandhealth.com

Biston Point of Care Bag Cleaning Guide

The Biston Point of Care Bag features a durable, water resistant coating on both the interior and exterior of the bag. This allows the bag to be wiped clean using antibacterial wipes or sprays. The bag also has rounded corners to ensure easy access cleaning.

Interior Cleaning

- Remove contents from the bag
- Remove examination mat (if applicable)
- Wipe clean all of the dividers and sections of the bags ensuring all surfaces are covered
- The dividers can be removed for further cleaning of the Velcro
- Remove the waste pod to ensure all surface can be wiped clean, inside and out
- Wipe clean the rear laptop storage section of the bag including the netting pocket and pen holders
- Allow to dry before use

Exterior Cleaning

- Ensure all zips are closed and then wipe clean the zip and zip pulls
- Unroll shoulder strap adjustments
- Wipe clean all surfaces
- Wipe clean the bag handle and identification panel
- Ensure mesh pockets are wiped thoroughly

Examination Mat Cleaning (if applicable)

- Remove examination mat from bags
- Wash examination mat at 60⁰ C
- Allow mat to dry before use