

Tristel™

daisy grip

Clean Clever Click.



Reusable Hygienic Tourniquet


STEP 1



APPLICATION

Loop daisygrip around the patient's limb, click magnetic clasp together.

STEP 2

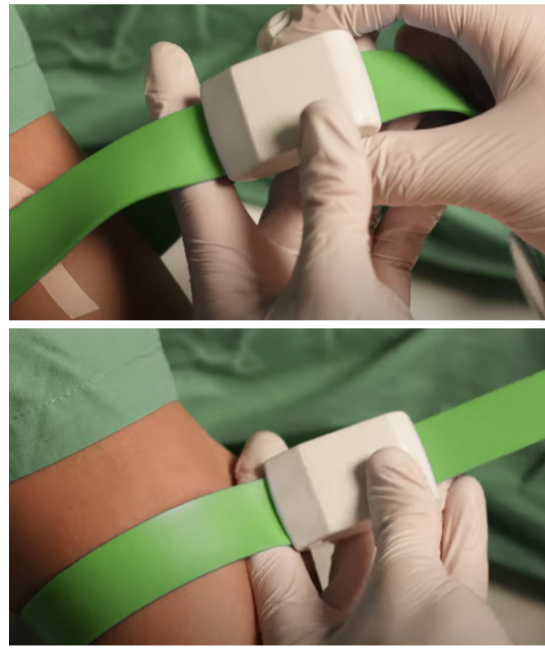

TIGHTENING

Place finger between strap and skin to prevent pinching. Pull strap from close to the clasp at a 45° angle to tighten – do not pull from the distal (tail end) of the strap.

STEP 3



LINE INSERTION/BLOOD COLLECTION

Loosen the daisygrip once the needle is in the vein.


STEP 4



LOOSENING

Place finger under clasp. Use a 'lifting' motion to flip the clasp in the direction of the strap to loosen daisygrip and release stasis.

STEP 5

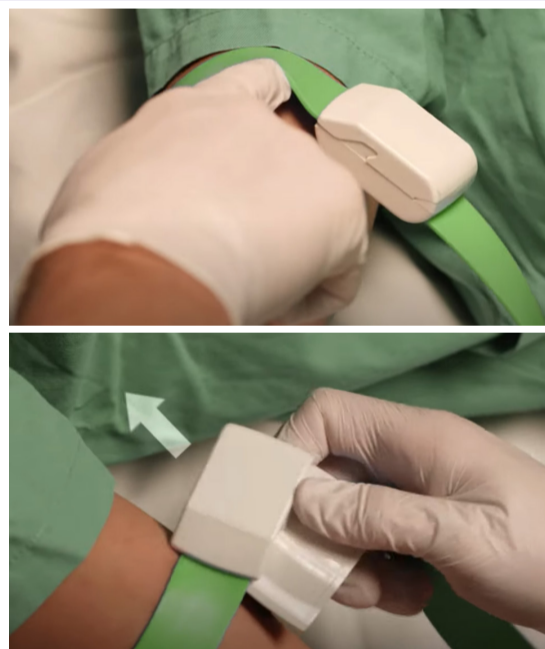

REMOVING

Loosen, then slide the top clasp sideways with your thumb, whilst holding the bottom arm of the clasp steady (with fingers or palm).

STEP 6



REPROCESSING

Ensure clasp and strap are cleaned and/or disinfected in line with local requirements after every use. Strap should be dry before next use.

Tristel™

**Medical Device
Decontamination**

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

Australia: Tristel Pty Ltd,
40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

New Zealand: Tristel New Zealand Limited,
P.O Box 15451, Tauranga, New Zealand
T +64 (0)7 5771560 - F +64 (0)7 5771567 - E mail-nz@tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions
Mkt-Wal-1981-1
15/JUN/2022

Tristel™

WE HAVE CHEMISTRY.