



SHEFFSHIELD

PRODUCT INFORMATION & INSTRUCTIONS



Adjustable PPE Face Shield

SheffShield is a PPE device designed to protect against airborne liquid droplets of substances and mixtures of harmful biological agents which are hazardous to health and cause illness. This device provides full face and lateral protection, featuring a liquid-proof, transparent visor to provide protection to the user.

Product Name: Full Face Visor

Product Model: 3366-010



Note:


- This protective face shield is manufactured for COVID-protection only—it is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19.
- The face shield may be used in conjunction with a respirator mask and/or additional eye protection such as glasses, and a gown.
- It is the responsibility of individual users to determine allergies to any materials used to construct this face shield; they should not use the product if allergic to the following:
Headband - **polypropylene**; Visor - **PET**; Strap - **silicone rubber**.
- This product is BSI Certified to PPE Regulation for Healthcare Professionals 2020/403.

To assemble:

1. **Ensure** the visor 'L' stamp is positioned on the user's left side when assembling;
2. **Push** the two front headband mounting studs through the middle two visor holes;
3. **Stretch** the headband arms outwards

ASSEMBLY

4. **Attach** elastic strap to the rear notch on each arm of the headband.

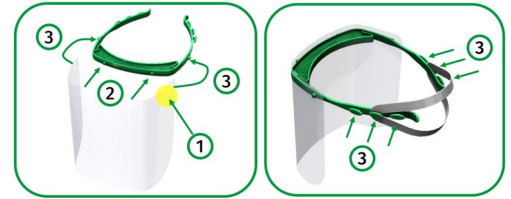


To fit:

1. **Hold** headband arm ends;
2. **Place** elastic at the back of the head;
3. **Pull** the headband forward into position;

FITTING


4. **Ensure** the elastic strap is at a comfortable position. If not, reposition elastic strap to a different notch.

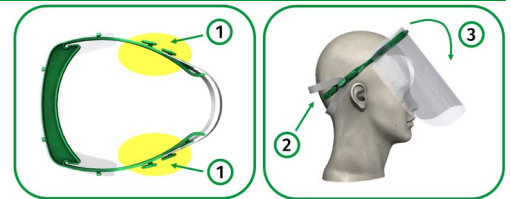

To remove:

1. **Hold** headband arm ends;
2. **Pull** the headband forward then up from the front of the face, releasing the elastic.

REMOVAL

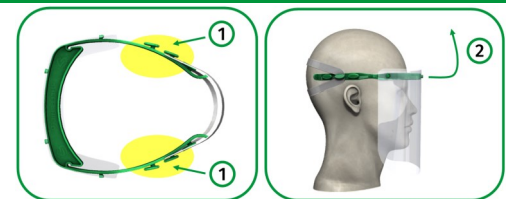

CLEANING & DISINFECTION

The face shield **can be cleaned and disinfected** using sodium hypochlorite, ethyl alcohol or isopropyl alcohol following suitable disinfection directions.

MAINTENANCE

Regular inspection of the face shield should be completed and if any defects found, the item **discarded**:

1. **Ensure** the visor surface is free from scratches which could dangerously impair vision.
2. **Ensure** the headband and retaining strap are damage and wear free.

The face shield has a shelf-life of 3 years after manufacture and after this date should be **retired and destroyed**.

STORAGE & TRANSPORT

The face shield **should be stored** in a dry location away from sources of heat and strong sunlight.

Do not store with any objects on top of the face shield, or by stacking the shield such that the headband may become permanently deformed.

Certification: BSI's PPE for Healthcare Professionals 2020/403—Eye protection Technical Specification
Notified Body: BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP, UK
Notified Body No.: 0086
BSI Certificate No.: 729058

For more information, please get in touch:
 Email: info@sheffshield.co.uk



Find us online



@SheffShield



facebook.com/SheffShield