


Sharps Protecta Pad User Guide


For the safe collection and disposal of used blades and needles.

The sequences shown here should be used as guidance. Please refer to local hospital guidelines for further information.

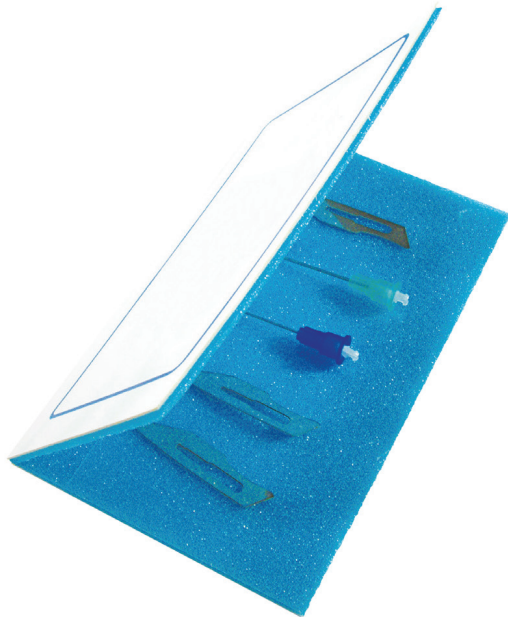

01

Remove the paper strip covering to reveal the blue foam pad with an adhesive surface.

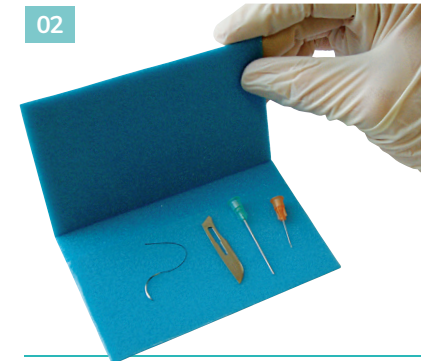

02

Place any discarded sharps onto the blue foam area of the pad using the appropriate instrument.



03

When the procedure is complete, fold the lid over at the perforated line and seal.



04

Dispose of the complete unit in the nearest sharps bin provided.



For more information:

Tel: 01530 278 111 | Email: info@365healthcare.com | www.365healthcare.com